

A camera sleeve system that offers 100% true sterility where others fail. The adaptor sleeves unique system allows safe, quick and easy application.

STEP 1



Simply insert your camera inside the telescopically folded sleeve utilizing applicator provided.

SPECIFICATIONS

Catalogue number DCC 8007M

Dimensions 140mm x 2850mm

Distal tip Endoscopic coupler

Packaging Sterile in packs of 40

Certification CE 0805

AS/NZS ISO 9002 : 1994

Latex Free

Patent Pending

STEP 2




Place enclosed camera over the eyepiece adaptor, utilizing the cameras own clipping mechanism. Pull back telescopically folded portion of the camera sleeve to cover the camera lead.

STEP 3



To complete the process, simply place the telescope into the C coupler mechanism, which ensures alignment and a secure fit. A locking mechanism is provided for optional prevention of telescope rotation.

Distributed by

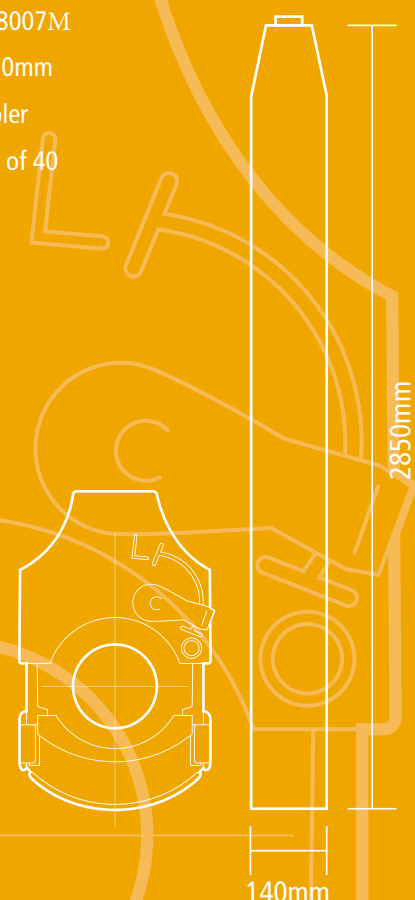

www.fairmontmedical.com.au

